



**TRY-IN RECOMMENDATIONS  
FOR  
ACCREDITATION TYPE CASES**

Dear Doctor:

Since this an AACD Accreditation, Fellowship or other similarly demanding case, we highly recommend you take into account the following recommendations:

1. DO NOT cement or permanently deliver any of the restorations at the try-in appointment
2. Place all restorations and review contacts, occlusion, interproximal closure, margin placement
3. Review esthetics and shade
4. Take photos of restorations in place: use isolating gel or temp cement if necessary to retain restorations correctly (\*see list below)
5. Make necessary adjustments to restorations and take another set of photos after adjustments are made
6. Put patient back in temps
7. Send photos to lab (Excel) and your consultant
8. Review photos with us and consultant
9. Return complete case (every single item) to lab for adjustments  
ie: impressions, implant abutments, all models, etc, etc
10. Do another try-in appointment same as above

Do not deliver restorations unless you are 100% satisfied.

Once cemented, restorations cannot be readily modified, and have to be remade, at Drs expense and responsibility

Most successful cases follow this protocol and undergo some degree of adjustment

-Over-

Recommended photographic views for try-in evaluation:

All AACD views, (except 11 if lower arch case or 12 if upper arch case  
AACD view No. 5 in occlusion, and same view No. 5 w teeth slightly parted  
Photos with relevant shade guides if there is a shade issue (At least 15 shots  
at different angles please, the more the better)

Profile view 10/1 and 2/1

Any other view that depicts issue to be addressed

Thank You

Charles Moreno, CDT, MDT  
AACD Accredited Master Ceramist  
1-800-981-9008