



# Chairside Guide

Full Contour Zirconia allows for conservative tooth preparation

## Classic preparation for all-ceramic crowns

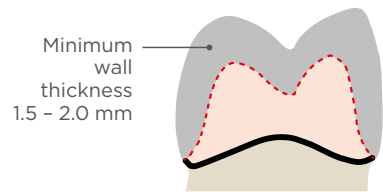


## Conservative preparation for full contour crowns

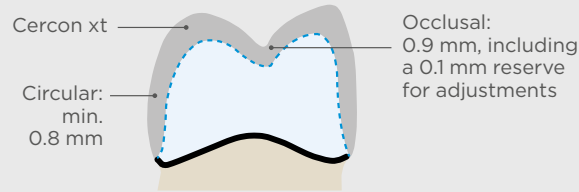


Six single crowns in the upper jaw from 6 - 11 (USA), 13 - 23 (FDI/International), made of Cercon xt extra-translucent zirconia.

Fully contoured crown made of Cercon xt, stained and glazed with Universal Stain & Glaze.

## Classic preparation for all-ceramic bridges

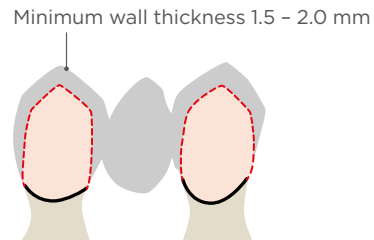

## Conservative preparation for full contour bridges

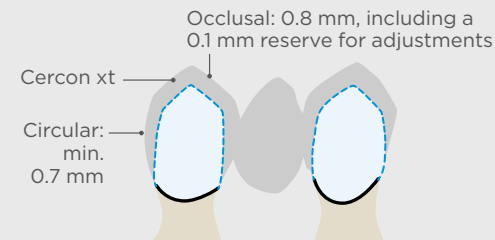

Two single crowns in the upper jaw from 13 - 14 (USA), 25 - 26 (FDI/International), made of Cercon xt extra-translucent zirconia.

Fully contoured crown made of Cercon xt, stained and glazed with Universal Stain & Glaze.

# Cementation/Bonding Recommendations

Cercon® restorations have a clinical history of success, with a wide-variety of cementation systems, offering the practitioner nearly unlimited choices in placing the final restoration.

## Calibra® Ceram Adhesive Resin Cement

The Maximum Strength Choice  
With Easy Cleanup

Calibra Ceram cement provides extra strength for enhanced clinical performance. No need to use a self cure activator, and no mixing of primers when used with Prime&Bond elect universal adhesive (Canada and USA), Prime&Bond active universal adhesive (Europe).



## Calibra® Universal Self-Adhesive Resin Cement

The Easy Universal Choice  
With Easy Cleanup

One step cementation means no additional etchant or adhesive step for a wide range of indications.



## Glass/Resin Ionomer Cementation

The use of resin modified glass-ionomer cements and glass-ionomer cements has been clinically proven.

# Polishing Recommendations\*

• **Axis\*\*\* Dental Adjustment & Polishing**  
Set for Full Contour Zirconia (LS-7579)

• **EDenta Ceramic Finishing Kit**  
(30055SO)

• **Brasseler USA®  
Adjustment Diamonds:**  
27 - 76 microns (837LF FG014)

**Polishing Diamonds:**  
Coarse/Medium/Fine (9545)

\*Always use water cooling when adjusting full contour zirconia restorations.

## Curing light recommendation

• **Smartlite Focus**



\*\*Axis®, EDenta & Brasseler USA® are not trademarks of Dentsply Sirona.